



KardiaMobile 6L
FDA-Cleared
6-lead personal ECG



KardiaStation
Application for recording an ECG
with KardiaMobile 6L





KardiaPro
Web-based portal to
access ECGs and
QT results

To record an ECG

1. Access KardiaStation app on your mobile phone or tablet
2. Select **"Record an EKG"**
 - Confirm that **"Six-Lead EKG"** is selected at the bottom of the screen
3. Enter MRN or Patient ID
4. Confirm device serial number and patient MRN
5. Instruct patient to place fingers or thumbs on the top left and right electrodes. The **"Λ"** symbol should point forward (as pictured) **A**
6. Select placement option for the electrode on the bottom of the 6L device:
 - if sitting up, above left knee **B**
 - if supine, lower left quadrant of abdomen **C**
7. ECG will begin recording when good skin contact is made on all 3 electrodes **D**
8. Patient must hold still during the 30-second ECG recording
9. If you get a message indicating the ECG was **"Interrupted"** or **"Unreadable"**, select **"Record Again"**
10. When finished recording, KardiaStation will display a cardiac rhythm assessment. If required, consult cardiology
11. Select **"Request QT Analysis"***
12. Select **"Done"** to complete
13. ECG recording is immediately available in KardiaPro. QT results will be available in KardiaPro in approximately 1 hour

*QTc Analysis is performed by an independent testing facility.
Skip this step if you are calculating QTc on your own.

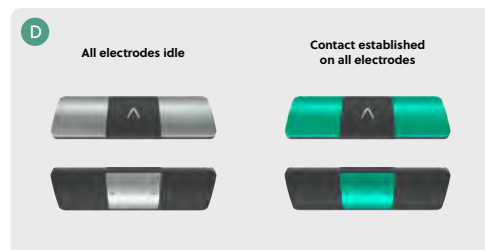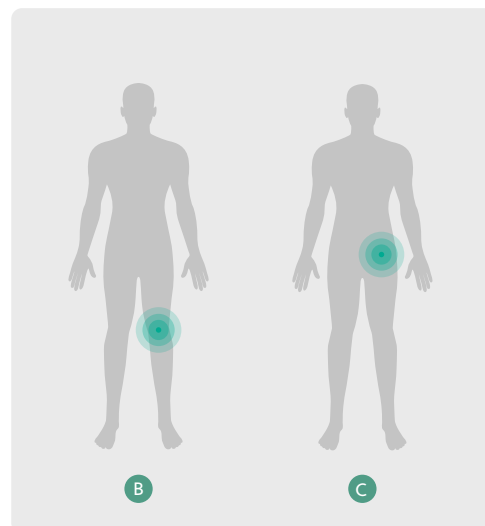

AliveCor Customer Support

For customer support or questions, please contact
support@alivecor.com or call (855) 338-8800.
Hours: Mon-Fri 8am-8pm ET, Sat-Sun 10am-7pm ET

KardiaMobile 6L is recommended for single patient use. Approved cleaners include soap water solution or bleach solution as recommended by the CDC. To clean: spray the cleaner on a soft cloth, and thoroughly wipe the device. Ensure device is sufficiently dried. Precaution: Do not use disinfecting wipes or alcohol based products as these products could adversely affect the product performance.

FDA Emergency Use

KardiaMobile 6L, in combination with KardiaStation and/or KardiaPro, can be used by healthcare professionals to monitor for QT prolongation in inpatients and outpatients receiving QT-prolonging medications, including hydroxychloroquine and azithromycin, whether alone or in combination, for the treatment of COVID-19. QT analysis results displayed in KardiaPro are adjunctive and should not be solely or primarily relied upon to treat COVID-19 or co-existing conditions.